

Sterile Wrap Cart

ASSEMBLY INSTRUCTIONS

CAUTION

INSPECT CONTENTS IMMEDIATELY AND FILE CLAIM WITH DELIVERING CARRIER FOR ANY DAMAGE. **SAVE YOUR BOX AND ALL PACKING MATERIALS.** YOU ARE RESPONSIBLE FOR DAMAGE TO YOUR UNIT IF RETURNED IMPROPERLY PACKED.



Parts List

qty	description	part number
1	.shelf	2448C or 2460C
4	.bolt	304980
4	.nut	310481
4	.casters	casters
2	.support bar frame	A203864
4	.support bar	A203900 or A203901
2	.collar clip	A203866
2	.collar clip	A204325
4	.posts	CP63-C
2	.three sided frame	B201242 or B201243

1 Casters & Shelf

Insert casters into posts. Assemble split sleeves on the first groove found on the bottom of the post. The split sleeves snap together around the posts. The wider end of the split sleeve faces the bottom of the assembly. Lower shelf down on posts until the split sleeves rest snugly in the shelf collars.

2 Three-sided Frame

Assemble split sleeves on the middle of the posts. The split sleeves snap together around the posts. The wider end of the split sleeve faces the bottom of the assembly. Lay the three-sided frame on top of the split sleeves.

3 Bottom Collar Clip

Assemble split sleeve on post so that the number "36" is showing above the split sleeve on the posts. The right-hand posts should have the collar slip with the clip facing the left-hand posts and the left-hand posts should have the clip collar with the clip facing the right-hand posts. Slide collar clips down onto posts with collars facing appropriate directions.

4 Top Collar Clip

Assemble split sleeve on post so that the number "58" is showing above the split sleeve on the right-hand posts and "58" on the left-hand posts. The right-hand posts should have the collar clip with the clip facing the left posts and the left posts should have the collar clip with the clip facing the middle posts. Slide collar clips down onto posts with collars facing appropriate directions.

5 Support Bar Frame

Attach right-side support bar frame to collar clips on inside of right-hand posts. Attach with bolt on inside of post and nut on outside. Attach left-hand support bar frame to collar clips on inside of outside (left) posts. Attach with bolt on inside of post and nut on outside.

6 Support Bars

Place support bar into support bar frame in 2nd slot. Place each additional bar into frame leaving approximately 5 slots between each bar.

7 Three-sided Frame

Assemble split sleeves on the top groove of the posts. The split sleeves just snap together around the posts. The wider end of the split sleeve faces the bottom of the assembly. Lay the three-sided frame on top of the split sleeves.

EG9878 Revised 05/20