

Catheter Procedure Cart - With Shelves

ASSEMBLY INSTRUCTIONS

For Catheter Procedure Cart Without Shelves, see page 3.

CAUTION

INSPECT CONTENTS IMMEDIATELY AND FILE CLAIM WITH DELIVERING CARRIER FOR ANY DAMAGE. **SAVE YOUR BOX AND ALL PACKING MATERIALS.** YOU ARE RESPONSIBLE FOR DAMAGE TO YOUR UNIT IF RETURNED IMPROPERLY PACKED.

(dolly truck model shown)

Parts List		
qty	part #	description
3	2424C or 2436C	chrome shelf
2	2460C	chrome shelf
4	303742	screws
4	310481	nuts
2	A203830	tote boxes
1	A203833	tote box slide
2	A203864	support bar frame
2	A203866	collar clip
2	A204325	collar clip
6	A203889	supply bins
16 or 24	A204176	catheter hooks
1	DF2460-A*	dolly truck
4	L24-4C	ledge
2	L60-4C	ledge
4	P63-C	posts
2	CATHP58-C	catheter posts
4	SB24 or SB36	support bars
6	A204331	shelf markers

* only on dolly truck model

1) Four Outside Posts and First Shelf

Line up the four outside posts with the four holes on the truck dolly. From underneath the truck dolly insert the hex head bolt into the post and tighten. Assemble plastic split sleeves on the posts so that the number 3 is showing above all. Lower shelf down onto posts until split sleeves rest snugly in the shelf collars.

OR

Casters & First Shelf

Insert casters into posts. Assemble split sleeves on the first groove found on the bottom of the post. The split sleeves just snap together around the posts. The wider end of the split sleeve faces the bottom of the assembly. Lay the first shelf down on top of the split sleeve.

2) Middle Posts

From the very right side of the bottom shelf count over 28 spaces, between the wires, and place the catheter post clip on the bottom shelf so that the prongs clip onto the wires running front to back. One clip should be on front and one on the back. Screw the two shorter posts onto the clips.

(continued on page 2)

Catheter Procedure Cart - With Shelves

ASSEMBLY INSTRUCTIONS

3) Second and Third Shelves

Assemble split sleeves on post so that the number 18 is showing above the split sleeve on the right post and 14 on the middle post. Lower second shelf down onto posts. Assemble split sleeves on post so that the number 33 is showing above the split sleeve on the right post and 29 on the middle post. Lower third shelf down onto posts. Make sure that all split sleeves fit snugly in the shelf collars.

4) Bottom Collar Clip and Fourth Shelf

Assemble split sleeves on post so that the number 33 is showing above the split sleeve on the middle posts and 36 on the left posts. The middle post should have the clip collar with the clip facing the left post, and the left post should have the clip collar with the clip facing the middle post. Slide collar clips down onto posts with collars facing appropriate directions. Assemble split sleeves on post so that the number 49 is showing on the right post and 45 on the middle posts. Lower fourth shelf down onto posts. Make sure that all split sleeves fit snugly in the shelf collars.

5) Top Collar Clip and Top Shelf

Assemble split sleeves on post so that the number 55 is showing above the split sleeve on the middle posts and 58 on the left posts. The middle post should have the collar clip with the clip facing the left post, and the left post should have the collar clip with the clip facing the middle post. Slide collar clips down onto posts with collars facing appropriate directions. Assemble split sleeves on top groove of left and right posts. Lower top shelf down onto posts. Make sure that all split sleeves fit snugly in the shelf collars.

6) Support Bar Frame

Attach right-side support bar frame to collar clips on inside of middle (right) posts. Attach with bolt on inside of posts and nut on outside. Attach left-side support bar frame to collar clips on inside of outside (left) posts. Attach with bolt on inside of post and nut on outside.

7) Support Bars

Place support bar into support bar frame in second slot. Place each additional bar into frame leaving approximately five slots between each bar.

8) Catheter Hooks

Snap three catheter hooks onto each support bar, space evenly.

9) Tray Slide and Tote Boxes

Attach tray slide to the underneath of the third right-hand shelf by hanging hooked part of tray slide to the frontmost part of the shelf. Slide tote boxes into tray slide with first one using top slides.

10) Ledges

Shelf ledges are to be placed on the top and bottom shelf. Insert leg (on ledge) with bend on inside of the edge of shelf. The other leg should be on the outside of the edge of shelf.

11) Supply Bins & Shelf Markers


Place all supply bins on the bottom shelf. Install shelf markers on the three right-hand shelves.



Catheter Procedure Cart - Without Shelves

ASSEMBLY INSTRUCTIONS

For Catheter Procedure Cart With Shelves, see front page.

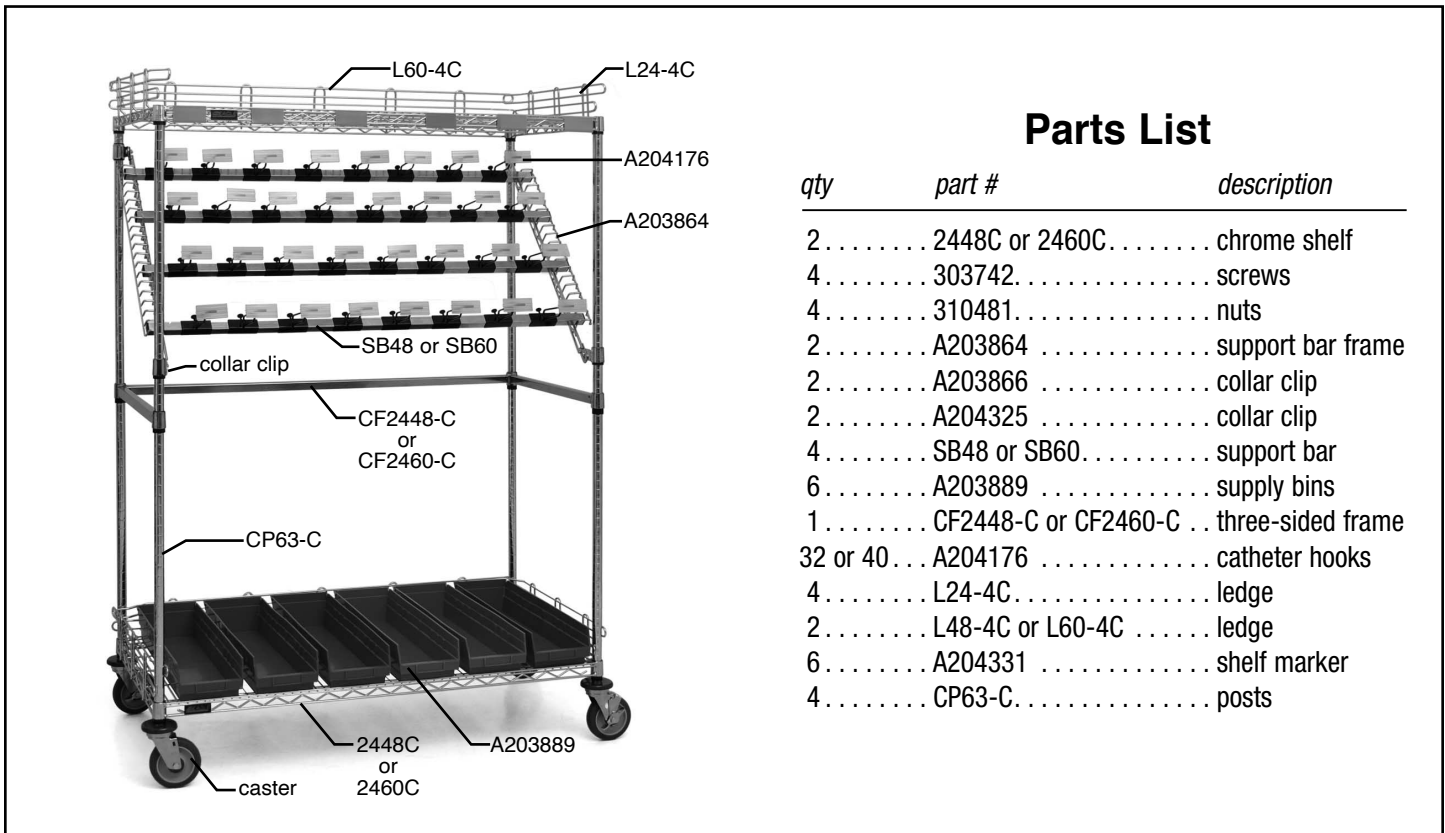

Parts List

<i>qty</i>	<i>part #</i>	<i>description</i>
2	2448C or 2460C	chrome shelf
4	303742	screws
4	310481	nuts
2	A203864	support bar frame
2	A203866	collar clip
2	A204325	collar clip
4	SB48 or SB60	support bar
6	A203889	supply bins
1	CF2448-C or CF2460-C	three-sided frame
32 or 40	A204176	catheter hooks
4	L24-4C	ledge
2	L48-4C or L60-4C	ledge
6	A204331	shelf marker
4	CP63-C	posts

1) Casters and First Shelf

Insert casters into posts. Assemble split sleeves on the first groove found on the bottom of the post. The split sleeves just snap together around the posts. The wider end of the split sleeve faces the bottom of the assembly. Lay the first shelf down on top of the split sleeve.

2) Three-sided Frame

Assemble split sleeves on the middle of the posts. The split sleeves snap together around the posts. The wider end of the split sleeve faces the bottom of the assembly. Lay the three-sided frame on top of the split sleeves.

3) Bottom Collar Clip

Assemble split sleeve on post so that the number 36 is showing above the split sleeve on the posts. The right-hand posts should have the collar clip with the clip facing the left-hand posts, and the left-hand posts should have the collar clip with the clip facing the right-hand posts. Slide collar clips down onto posts with collars facing appropriate directions.

4) Top Collar Clip

Assemble split sleeve on post so that the number 58 is showing above the split sleeve on the right- and left-hand posts. The right-hand posts should have the collar clip with the clip facing the left-hand posts, and the left-hand posts should have the collar clip with the clip facing the right-hand post. Slide collar clips down onto posts with collars facing appropriate directions.

(continued on back page) 

Catheter Procedure Cart - Without Shelves

ASSEMBLY INSTRUCTIONS

5) Top Shelf

Assemble split sleeves on the top groove of the post. The split sleeves snap together around the posts. The wider end of the split sleeve faces the bottom of the assembly. Lower the top shelf down onto the posts until the split sleeves rest snugly in the shelf collars.

6) Support Bar Frame

Attach right-side support bar frame to collar clips on inside of right posts. Attach with bolt on inside of post and nut on outside. Attach left support bar frame to collar clips on inside of outside (left) posts. Attach with bolt on inside of post and nut on outside.

7) Support Bars

Place support bar into support bar frame in second slot. Place each additional bar into frame leaving approximately five slots between each bar.

8) Catheter Hooks

Snap three catheter hooks onto each support bar, space evenly.

9) Ledges

Shelf ledges are to be placed on the top and bottom shelf. Insert leg (on ledge) with bend on inside of the edge of shelf. The other leg should be on the outside of the edge of shelf.

10) Supply Bins & Shelf Markers

Place all supply bins on the bottom shelf. Install shelf markers on the top and or bottom shelves.

(End)



• 100 Industrial Boulevard, Clayton, Delaware 19938-8903 U.S.A.
• Phone: 302/653-3000 • 800/441-8440
• Fax: 302/653-2065 • www.eaglegrp.com

Eagle Foodservice Equipment, Eagle MHC, and SpecFAB® are divisions of Eagle Group. ©2023 by the Eagle Group